nimo®

NEBULİZER SET

NEBULIZER SET

FTALAT
İÇERMEZ
FTALAT
FREE

TR - PEDIATRİK
EN - PEDIATRIC

REF 099-142

LOT

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CAUTION

- 1- Do not use damaged package. Suitable for single person use.
- 2- Do not stock in high temperature, sun shining and moisture place.
- 3- There is not ftalat and latex in this product.

INSTRUCTION FOR USE

- 1- Install the air-tube to air out hole on the device.
- 2- Install the other edge of air-tube to down side of medication cup.
- 3- Turn the medication cup lid reverse of clockwise and pull up and take out it.
- 4- Put the right type and amount of medication in to medication cup as your doctor advised to you.
- 5- Place the vaporiser head under the lid of the medication cup so that the tabs engage.
- 6- Put the medication cup on its place and turn clockwise the lid to close it.
- 7- As your usage choice install the mask or mouthpiece on to medication cup. Operate the device, Be sure aerosol is coming from medication cup.
- 8- Treatment with mask, wear the mask to cover your mouth and nose and keep breathing normally.
- 9- Treatment with mouthpiece, put the mouthpiece between your lips and teeth and keep breathing normally. (Valid for mouthpiece.)
- 10- Adjust the desired aerosol output using the nebulization control valve located on the lid of the medication cup.

Iss. Date 28.11.2019 Rev01